## NOR Brukerveiledning

### Brukerveiledning for myke linser

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## CE 0459 (Year of authorisation of the affixing: 2011)

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## GB Recommendations for use Guide to the use of your soft lenses

This leaflet contains important information; read it and keep it for future reference.

Lenses are medical devices and must always be adjusted by a contact lens specialist. For correct use, it is important that you follow all the instructions given by your specialist and all the recommendations given on the packaging of your lenses and maintenance products.

**INSTRUCTIONS:**

Soft lenses are corneal lenses for daytime wear, changing them every three months, six months or once a year, to correct short-sightedness, long-sightedness, astigmatism and presbyopia on regular corneas.

In children and young people under 20 years old, AMYOPIC contact lenses can slow down the progression of myopia.

**RECOMMENDED RENEWAL:**

Every three months, every six months or once a year, as applicable.

**PACKAGING:**

The lenses are packaged in a sterile bottle containing a saline solution. The bottle is fitted with a tamper-proof system until first opening. If packaging is damaged, do not use the lens.

**The label provides the following information:**

<span></span>	The lens brand name
<span></span>	The name of its material
<span></span>	The name and address of the manufacturer
<span></span>	The serial number
<span></span>	European conformity with health and safety requirements and certifying organisation number
<span></span>	Expiry date
<span></span>	Steam sterilisation
<span></span>	Leaflet consultation
<span></span>	Do not open if the packaging is damaged

#### PRECAUTIONS FOR USE

Your specialist will give you precise instructions on how to look after your lenses, put them in and take them out.

Always wash and dry your hands thoroughly before putting your lenses in or taking them out.

Never use damaged lenses.

After use and before re-use, the lenses must be taken out, cleaned and decontaminated using the maintenance products your specialist has recommended.

These lenses are designed to be worn by day; put them in the morning and take them out at night.

The minimum recommended time for wearing AMYOPIC lenses for them to be effective is 10 hours a day or more depending on physiological tolerance.

Always take your lenses out before going to bed.

If you do not wear your lenses for several days, refer to the information about your maintenance solution before re-using them. Always make sure you have your glasses with you (in case you find yourself obliged to take your lenses out).

**To avoid any alteration to the material or shape of your lenses, make sure you change them every 3 months, 6 months or year, as indicated on the lens bottle.**

**Make sure that you attend the scheduled appointments with your practitioner; these are essential to monitor any changes.**

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