F

PRECILENS B5

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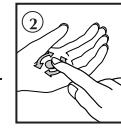
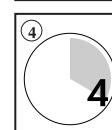
UK

PRECILENS B5

All in one solution for RGP contact lenses

PRECILENS B5 All in One solution

For cleaning, removing of protein deposits, rinsing, disinfecting, storing, inserting, moistening and eye care... All in one. For RGP contact lenses.

MEDICAL DEVICE

Composition:

Active Ingredient: Poloxamine 0,5%, Dexpanthenol 1,0%, HydraFlex-RGP 0,8%.
Preservative: PHMB 0,0001%.

Read the instructions and warnings carefully for proper use of the PRECILENS B5 in combination with your contact lenses.

Precautions and instructions for use:

Wash your hands carefully before removing the lenses from your eyes or the lens case.
Dry your hands using a clean towel.

Unscrewing the cap from the bottle will break the guarantee seal.

Immediately after removal of the lenses, clean them with a few drops PRECILENS B5.
Clean the lenses by carefully rubbing them with your index finger in the palm of your other.

Rinse the lenses with PRECILENS B5.

To disinfect the lenses, place them in the lens case filled with fresh PRECILENS B5 and leave to stand for at least 4 hours.

After disinfecting, rinse both sides of the lens with PRECILENS B5.

The PRECILENS B5 -moistured lenses can then be inserted.

Rinse the lens case with PRECILENS B5 (or a Saline solution) and allow the lens case to dry in the air.

Moisturizing: if you are bothered by minor irritation while wearing the lenses, apply a drop of PRECILENS B5 to the eye and blink several times. You can leave your lens in the eye. If problems continue, consult your contact lens specialist.

Do not touch the dropper tip of the bottle as this may contaminate the solution.

Close the bottle tightly immediately after use.

Other instructions:

After storage in excess of 7 days, lenses must be cleaned and disinfected again before wear.

Replace your lens case frequently to prevent infection due to contamination.

Warnings:

Do not use if guarantee seal is damaged.

Do not mix or use PRECILENS B5 with other contact lens fluids, unless it is indicated in the instructions for use.

Always replace any remaining solution in the lens-case with fresh solution.

Do not use after the expiry date.

Do not change your contact lens system without consulting your doctor / contact lens specialist.

Never use tap water as an alternative for this contact lens fluid.

Do not use:

If you are prescribed eye medication, consult your ophthalmologist or contact lens specialist before using the medication in combination with contact lenses.

In case of eye irritation, discontinue use and consult your doctor/contact lens specialist.

Store:

Keep out of the reach of children.

Store between 4°C and 25°C (40°F and 77°F).

Protect from sunlight.

Keep the solution in its original packaging.

Use within 2 months of opening.

Side effects:

Do not use PRECILENS B5 All in One solution + B5 if you are allergic to one of the ingredients.

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